

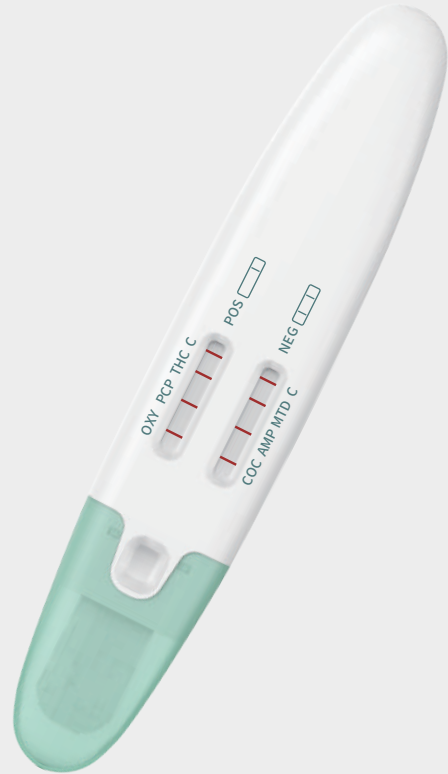


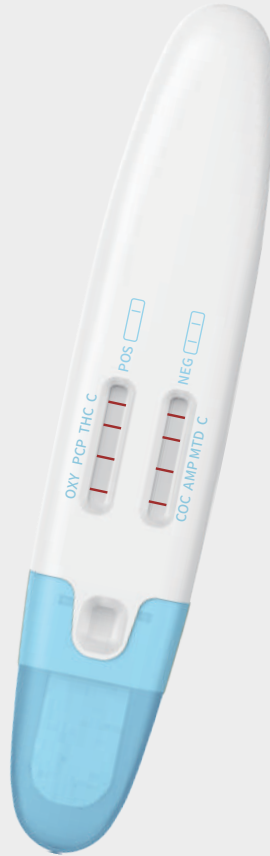

Multi-Drugs Rapid Test Device (Oral Fluid)

Multi-Drugs Rapid Test Device (Oral Fluid)

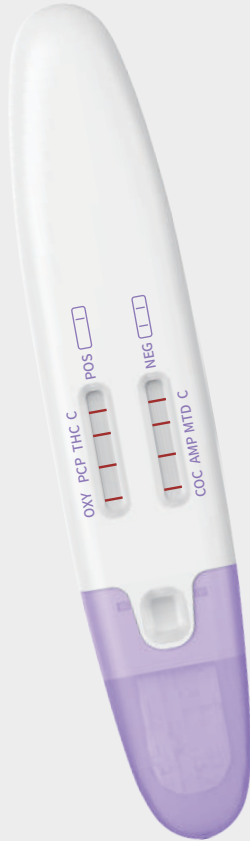

All-In-one midstream, screen for up to 14



Simple operation, no equipment required




High accuracy and privacy protected




▼ Test Procedure

1 Remove the cap

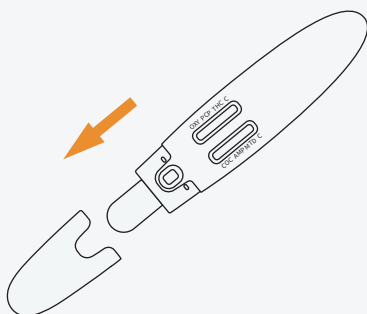

2


Under your tongue for 1-3 minutes, take the absorbent tip out from the mouth when the red color moves across the result window in the center of the device.

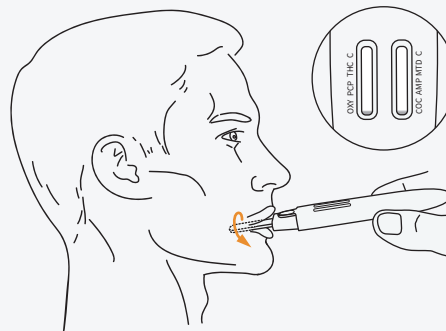

3




Read ALC result
at 2 minutes

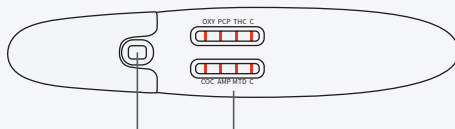


Read Drugs result
at 5 minutes

▼ Test Results

Drug test
results

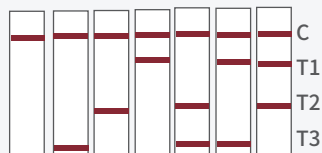

— Preliminary Positive —

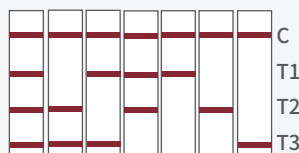

— Negative —



— Invalid —

Alcohol test
results



BAC

0
0

0.02%
20mg/dL

0.04%
40mg/dL

0.08%
80mg/dL

0.30%
300mg/dL



▼ Products Information

Specimen	Item	Cut off (ng/mL)	Item	Cut off (ng/mL)
Oral Fluid	Alcohol (ACL)	0.02%	Tenamfetamine(MDA)	50
	Amphetamine (AMP)	50	MDMA	50/100
	Secobarbital (BAR)	50	MDPV	300
	Buprenorphine (BUP)	5/10	Methamphetamine (MET)	25/50
	Oxazepam (BZO)	30/50	Morphine (MOP/OPI)	20/40
	Cocaine (COC)	20	Methadone (MTD)	30
	Cotinine (COT)	50	Oxycodone (OXY)	20
	EDDP	20	Phencyclidine (PCP)	10
	Fentanyl (FYL)	10/20/50	Nortriptyline (TCA)	100
	Synthetic Marijuana (K2)	20/25	Marijuana (THC)	40/50
	Ketamine (KET)	50/100	Tramadol (TRA)	30



VENDOR CODE & ITEM	UOM
SALIVASTIK-254 Saliva Stik Oral Fluid Device 5 Panel AMP, COC, OPI, MET, THC	CS25
SALIVASTIK-364 Saliva Stik Oral Fluid Device 6 Panel AMP, COC, OPI, MET, BZO, OXY	CS25
SALIVASTIK-164A Saliva Stik Oral Fluid Device 7 Panel AMP, COC, MET, OPI, PCP, THC, ALC	CS25
SALIVASTIK-17105 Saliva Stik Oral Fluid Device 9 Panel AMP, BAR, BUP, BZO, COC, MET, OPI, OXY, PCP	CS25
SALIVASTIK-17104 Saliva Stik Oral Fluid Device 10 Panel AMP, BAR, BUP, BZO, COC, MET, OPI, OXY, PCP, THC	CS25
SALIVASTIK-4114 Saliva Stik Oral Fluid Device 12 Panel AMP, PCP, BUP, BZO, COC, THC, MET, MTD, OPI, OXY, FEN, ALC	CS25
SALIVASTIK-3114 Saliva Stik Oral Fluid Device 12 Panel AMP, BAR, BUP, BZO, COC, MET, MTD, OPI, OXY, PCP, THC, ALC	CS25
SALIVASTIK-6124 Saliva Stik Oral Fluid Device 12 Panel AMP, BAR, BUP, BZO, COC, MDMA, MET, MTD, OPI, OXY, PCP, THC	CS25
SALIVASTIK-2134 Saliva Stik Oral Fluid Device 13 Panel AMP, BUP, BZO, COC, MET, MTD, OPI, OXY, PCP, THC, FEN, ALC, K2	CS25
SALIVASTIK-946 Saliva Stik Oral Fluid Device 4 Panel AMP, COC, OPI, PCP	CS25
SALIVASTIK-986 Saliva Stik Oral Fluid Device 8 Panel AMP, BAR, BZO, COC, MTD, OPI, PCP, PPX	CS25
SALIVASTIK-956 Saliva Stik Oral Fluid Device 5 Panel AMP, COC, OPI, PCP, THC	CS25
SALIVASTIK-4109 Saliva Stik Oral Fluid Device 10 Panel AMP, BAR, BZO, COC, MET, MTD, OPI, OXY, PCP, THC	CS25
SALIVA-COT Saliva Stik Single Panel COT	CS25

For orders, call (866) 526-2873 or email sales@americanscreeningcorp.com